



AFB Schliessa 12, FL-9495 Triesen

Implants and BEMERtherapy

The BEMER 3000 therapy systems are certified class 2a medical products whose use poses no potential threat to the human organism. Implants of each type, [Endoprosthesis - Metal and plastic stents, heart valve replacements etc.] represent no contraindication for BEMERtherapy.

The magnetic intensity of the BEMER 3000 lies within the range of the flux density of the geomagnetic earth field (approx. 45 μ T), i.e. it lies within the strength of natural magnetic fields around us. It is well below the limit value prescribed by the WHO (100 μ T). Therefore, in actual fact, no side effects or contraindications are possible. Based on these very slight intensities, no relevant exertion of influence according to physical regularities (i.e. warming and/or ionization) on implants in the human organism is possible.

Therapy with BEMER 3000 extremely low, wide frequency range electro magnetic fields is a complex therapy method that stimulates the metabolism in general, allowing optimal regulation.

Above all, the improved microcirculation of the blood, the activation of "repair proteins" (HSP 70), the activation of enzyme reactions, the formation of Callus cells, all positively affect the linking of the implant into the surrounded tissue. This has been confirmed by European Multi centric medical user studies conducted by the *AFB Academy for Bioenergetics* that found 50% of Endoprosthesis patients experienced a strengthening of their condition.

Contraindications

Relative Contraindications for BEMERtherapy:

- Fresh organ transplantation (less than two years). We know that the BEMER strengthens the immune system. *In theory therefore, rejection is conceivable.* On the other hand the electromagnetic field improves the effectiveness of medicines, including those relating to immune suppressants. This is why we agree under medical supervision that the BEMER system supports possible organ transplantation. In isolated cases such BEMERtherapy has been carried out successfully. However, there are not enough reliable therapy experiences on hand. The use of the BEMER 3000 in the case of organ transplantation is thus only to be carried out at the practitioners discretion and responsibility

Only to be treated by a qualified practitioner skilled in BEMERtherapy:

Severe cardiac dysrhythmia; severe tachycardia; large aneurysms (from severity of the 3° – only use Intensity levels 1 -3); severe psychoses; acute heavy internal haemorrhage; temperatures above 38.5°; and non-compensated severe attack conditions.

The other contraindications known to occur in magnetic field therapy do not apply to the BEMER 3000 system due to its low flux density. In the case of active implants, e.g. pacemakers, a certain residual risk can never be fully excluded. Therefore ask your doctor for the permissible limit values for your implant.